PHYSIPRO[®]

Owner's manual

This manual contains important informations about this product. Please hand to the owner upon delivery.

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ENGLISH**INTRODUCTION**

Physipro Inc. is proud to count you among its customers and would like to thank you for the confidence you have shown by purchasing our product.

This owner's manual was created to provide you with all the information needed to allow you to use our product in a safe and optimal way. For all adjustments and settings, we strongly recommend you contact your distributor.

Physipro Inc., relieves itself of all liability should physical injury or property damage occur due to the lack of maintenance or misuse of our product, or from modifications made to the product without the prior written consent of Physipro Inc.

For Physipro Inc., your satisfaction is our highest priority.

PRODUCT INFORMATION

Date of purchase: _____

Supplier: _____

Address: _____

Telephone: _____

The product presented in this manual may not be an exact representation of your product. However, the instructions provided in this manual are still valid.

Physipro Inc. reserves the right to modify or change this manual without notice.

TABLE OF CONTENT

DESCRIPTION.....	3
COMPOSITION.....	3
WARNINGS	4
RECOMMENDATIONS.....	5
INSTALLATION	5
MAINTENANCE.....	6
WARRANTY.....	7
FRANÇAIS	8

CONTOUR INSERT™

The Contour Insert™ is composed of a firm base which features a recess in the pelvic loading area for the addition of the insert of your choice. The Contour Insert™ seat cushion can be purchased with different inserts and they can be used interchangeably to provide the pressure relief needed to continually ensure optimal comfort and skin protection.

This item meets the requirements of California 116 & 117, NFPA 701, CE marking and NF EN1021-1, EN1021-2 norms.

COMPOSITION

Foam base

Top layer: Injected urethane (medium-firm)

Bottom layer: EVA foam

Cover(s) of your choice

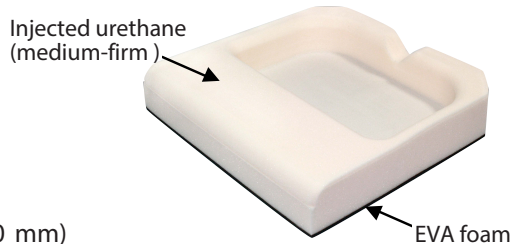
Available Sizes

Width: 14" - 24" (356 mm – 610 mm)

Depth: 14" - 24" (356 mm – 610 mm)

Thickness: 3" (75 mm)

Weight capacity: 14" to 20" (356 mm to 508 mm) width: 300 lb
21" to 24" (533 mm to 610 mm) width: 500 lb

INSERT OPTIONS AVAILABLE

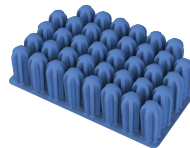
Soft Memory Foam
and Urethane

Temper Foam T36 and T38

Cubed Gel and Urethane

Air Cell Insert

WARNINGS

Consult a healthcare professional: Physipro™ recommends that a health care professional such as a doctor or therapist, experienced in seating and positioning, be consulted to determine, and install the required seating system and make any necessary adjustments.

Center of gravity: Installing a cushion on a wheelchair may affect the wheelchair's center of gravity and may cause the wheelchair to tip backwards, potentially resulting in injury. To ensure optimal stability of your wheelchair after the addition of a new cushion, always assess for the potential need for anti-tips or amputee axle adapter brackets to be added or adjusted to increase stability.

Cushion dimensions: The dimensions of the cushion must be adapted to the user and the wheelchair, if not, the cushion's benefits will be reduced or eliminated.

Cushion and cover compatibility: The cushion cover must be well-fitted and used as directed in this manual to prevent the formation of skin lesions. An oversized or undersized cover will reduce or eliminate the effectiveness of the cushion and can create an unstable surface, which can lead to a fall.

Environment: Avoid exposing foam to fluids or moisture for prolonged periods. Avoid exposing the cushion to extreme cold or heat. Never place cushion near open flame.

Instability/Fall risk: Never place other products or materials (except those specified in this manual) between the user and the cushion or underneath the cushion, doing so, will cause instability and increase the risk of injury from a fall.

Cushion positioning: Always refer to the orientation indicators of the cushion and the cover to properly position cushion on wheelchair. Proper positioning is essential to ensure maximum comfort and pressure relief.

Skin inspection: Cushion should be tested before prolonged use. To avoid the appearance of skin redness due to pressure points, Physipro™ recommends shifting your weight as frequently as possible and performing a pressure relief technique at least once every two (2) hours. You should regularly check (at least once a day) for skin redness especially around bony prominences. If skin redness or discoloration develops, discontinue use of the cushion. If discoloration does not disappear within thirty (30) minutes consult a healthcare professional immediately. Be aware that no cushion can substitute the need for proper skin hygiene, a balanced diet and frequent position changes.

Transport: Crash tests have not been conducted on these products and even if the mobility aid in which it is installed has obtained the ISO-717619, it will not be maintained.

RECOMMENDATIONS

Before use, verify the quality and condition of the cushion components.

To ensure your cushion provides optimal protection and comfort and to extend your product lifespan, the following precautions should be applied.

- Physipro™ seat cushions are reserved exclusively for use on mobility aids (manual or power wheelchairs), do not use cushion on other surfaces.
- Before installation, make sure the cushion is compatible with the wheelchair seat dimensions.
- Cushion must be installed on a good quality flat wheelchair surface. Surface must be cleaned before cushion installation.
- Verify height of armrest. Adjustments should be made, if needed.
- Never place a towel, a blanket, a folded bed sheet or any other material between the user and the cushion, this will diminish the cushion efficiency and increase the risk of developing skin lesions.
- It is preferable to wear clothes made of cotton, that are not too tight and do not have seams on pressure points.
- Before use, let the cushion get to room temperature to limit discomfort caused by excessively hot or cold surfaces.
- Never place cushion near sharp object.
- Remove any foreign object (crumbs, debris, tubing, etc.) from cushion surface.
- Clean on a regular basis and visually inspect all components of the cushion periodically.

It is recommended to consult your therapist in the month following the installation of a new cushion to make sure it is efficient.

INSTALLATION

Your cushion is assembled according to your specifications before delivery.

Simply position the cushion on the wheelchair, make sure the orientation indicator marked « **Front** » on the bottom of the outer cover is placed towards the front and facing downwards. Make sure the stitches on the cover are aligned with the cushion's edges.

Note: Velcro fasteners are provided for flat installation. Self-adhesive Velcro strips are attached to two fixed Velcro strips, located on the bottom side of the cushion's outer cover. To securely fix the cushion on a rigid seat pan, remove the protective film of the self-adhesive Velcro strips, position cushion and then press down firmly on the cushion to ensure Velcro has properly adhered to the seat pan.

ENGLISH

MAINTENANCE

Regular cleaning eliminates the accumulation of dirt and grime and may help extend the life of your cushion. It is recommended to visually inspect the components of the cushion during the cleaning process.

Cleaning the outer cover:

1. Remove cover from the foam base and fasten the zipper.
2. Machine wash in cold water on the gentle cycle or hand wash with a mild detergent.
3. Allow to air dry completely.
4. Reinstall cover on the foam base.

Do not iron.

Cleaning the incontinence cover: Hand wash with soapy water and use a clean dry cloth to gently wipe off excess moisture, then let air dry completely.

Cleaning the foam base:

1. Remove the outer cover and the incontinence cover and the insert.
2. Wipe lightly with a damp cloth, without soap or detergent.
3. Use a clean dry cloth to gently press on cushion to absorb excess moisture, then let air dry completely.
4. Reinstall cover(s) on the foam base.

Do not use soap.

Do not submerge in water.

Never place foam base in washer or dryer.

Cleaning your insert

Always remove your insert from the foam base for cleaning.

Depending on your insert's composition, follow the instructions below.

Cleaning the gel layer

1. Wipe lightly with a damp cloth, with water and a mild detergent.
2. Use a clean dry cloth to gently wipe off excess moisture.
3. Let air dry completely and install insert into the foam base.

Cleaning the foam layer

1. Wipe lightly with a damp cloth, without soap or detergent.
2. Use a clean dry cloth to gently press on foam to absorb excess moisture.
3. Let air dry completely and install insert into the foam base.

Cleaning the Air Cell insert

Please refer to the operation manual provided with your insert.

Storage

Physipro Inc. cushions should be stored on a flat surface, in a dry area and out of direct sunlight.

Life Span

The estimated life span of your cushion is three (3) years.

Disposal of Medical Devices

Do not burn or throw away medical devices outside of designated areas.

WARRANTY

The warranty period for the following part(s) and component(s) of your cushion are as stated below:

- Foam base: Twenty-four (24) months
- Cover(s): Six (6) months
- Soft memory foam and urethane insert: Twenty-four (24) months
- Temper foam T36 and T38 insert: Twenty-four (24) months
- Cubed gel and urethane insert: Twenty-four (24) months

This limited warranty covers defects in material and workmanship on all the standard or optional parts and components of this product, from the date of shipment, for the duration stated above.

Parts and components that are replaced or repaired under one of these warranties will remain subject to the warranty for the remainder of the warranty period. Physipro Inc. agrees to repair or replace, free of charge, any defective part or component for the full duration of the warranty period.

This warranty does not apply to:

- a) Products that were misused, neglected or have not been maintained and stored in accordance with the recommendations of the manufacturer.
- b) Products that were intentionally or accidentally altered, or products that have been damaged through repairs or modifications made without the prior written consent of Physipro Inc. or of an authorized retailer.
- c) Normal wear and tear or damages caused by the user such as punctures, tears, or burns.

Warranty service

If a defect is detected during the warranty period, please contact the retailer from whom you purchased your Physipro Inc. product as soon possible. Please refrain from using your product until an inspection has been made by a qualified technician.

In the event you do not receive satisfactory warranty service, please write directly to Physipro Inc. at the address listed below or send us an e-mail at order@physipro.com, please include the retailer's name and address, a brief description of the defect, the invoice date, and the products serial number, if applicable. DO NOT RETURN product prior to receiving a written return authorization from Physipro Inc.

The defective unit or parts must be returned for warranty inspection within thirty (30) days of the return authorization date at the following address:

Physipro Inc.
370, 10th Avenue South, Sherbrooke, Quebec, J1G 2R7

Exclusions and limitations

Compliance with the safety precautions and recommendations mentioned in this manual is crucial to avoid injury to the user or other persons. Inadequate or inappropriate adjustments will increase the risk of accident. It is important to thoroughly read the manual before making any adjustments. Physipro Inc. will not be held responsible for bodily injury or property damage caused by misuse or incorrect adjustments made to its product by an unqualified person.

Physipro Inc. disclaims all liability for damages that occur during transportation or are the result of situations beyond its control.

The foregoing warranty is exclusive and in lieu of all other expressed warranties.

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