PHYSIPRO[®]

Owners manual

This manual contains important informations about this product. Please hand to the user upon delivery.

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INTRODUCTION - ENGLISH

Physipro Inc. is proud to count you among its customers and would like to thank you for the confidence you have shown by purchasing our product.

This user manual was created to provide you with all the information needed to allow you to use our product in a safe and optimal way. For all adjustments and settings, we strongly recommend you contact your distributor.

Physipro Inc., relieves itself of all liability should physical injury or property damage occur due to the lack of maintenance or misuse of our product, or from modifications made to the product without the prior written consent of Physipro Inc.

For Physipro Inc., your satisfaction is our highest priority.

PRODUCT INFORMATION

Date of purchase: _____

Supplier: _____

Address: _____

Telephone: _____

The product presented in this manual may not be an exact representation of your product. However, the instructions provided in this manual are still valid.

Physipro Inc. reserves the right to modify or change this manual without notice.

CONVENTIONAL BACKREST

The Conventional rigid modular backrest is custom-made for each client using their anthropometric measurements. The Conventional backrest helps improve posture by preventing pelvic tilt. The lateral supports increase the user stability and balance and can be used in an effort to control spinal scoliosis. The backrest is attached to the backpost with a system of hooks and brackets that allow height, depth and angle adjustments. The Conventional rigid modular backrest provides postural correction and comfort.

Available in adult and pediatric models.

This item meets the requirements of California 116 & 117, NFPA 701, CE marking and NF EN1021-1, EN1021-2 norms.

SPECIFICATIONS

Back plate

Aluminium or Plastic


Standard backrest padding

Plastazote, Medium density memory foam and urethane

Mounting hardware

Hook and Bracket system

1. Mounting bracket
2. Swing-away stabilizer
3. Mounting hook
(2 or 5 adjustment holes)
4. Fixed stabilizer
5. Bracket anchor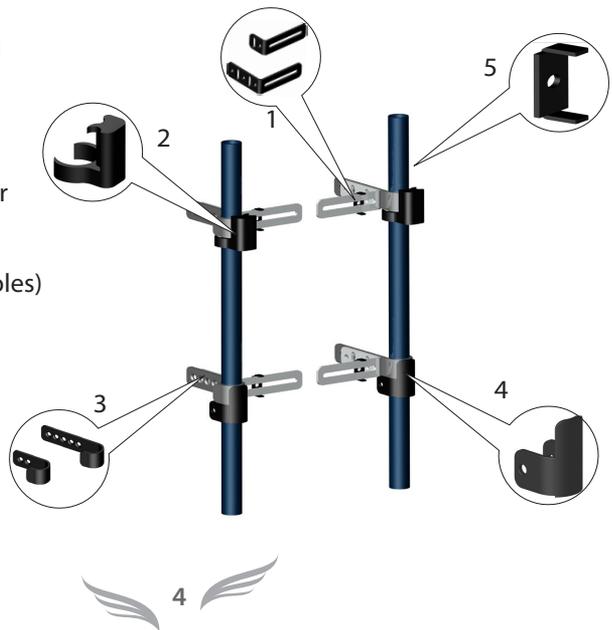

RECOMMENDATIONS

To ensure your backrest provides optimal protection and comfort and to extend your product lifespan, the following precautions should be applied.

- Before use, it is important to verify the quality and condition of the backrest components, please refer to Maintenance guide in section Maintenance, page 12.
- Physipro Inc. backrest are reserved exclusively for use on mobility aids (manual or power wheelchairs).
- Always install backrest with the provided mounting hardware.
- Never push or lift wheelchair with backrest, doing so may cause the backrest to detach from the wheelchair and can result in injury or damage.
- Before installation, make sure the backrest is compatible with the wheelchair's dimensions.
- Never place a towel, a blanket, a folded bed sheet or any other material between the user and the backrest, this will diminish the backrest efficiency and increase the risk of developing skin lesions.
- It is preferable to wear clothes made of cotton, that are not too tight and do not have seams on pressure points.
- Before use, let the backrest get to room temperature to limit discomfort caused by excessively hot or cold surfaces.
- Remove any foreign object (crumbs, debris, tubing, etc.) from backrest surface.
- Clean on a regular basis and visually inspect all components of the cushion periodically.
- It is recommended to consult your therapist in the month following the installation of a new cushion to make sure it is efficient.

WARNINGS

Consult a healthcare professional: Physipro Inc. recommends that a health care professional such as a doctor or therapist, experienced in seating and positioning, be consulted to determine, and install the required seating system and make any necessary adjustments.

Center of gravity: Installing a backrest on a wheelchair may affect the center of gravity of the wheelchair and may cause the wheelchair to tip backwards, potentially resulting in injury. To ensure optimal stability of your wheelchair after the addition of a new backrest, always assess for the potential need for anti-tippers or amputee axle adapter brackets to be added or adjusted to increase stability.

Backrest dimensions: The dimensions of the backrest must be adapted to the user and the wheelchair, if not, the backrest's benefits will be reduced or eliminated.

Backrest and cover compatibility: The backrest cover must be well-fitted and used as directed in this manual to prevent the formation of skin lesions. An oversized or undersized cover will reduce or eliminate the effectiveness of the backrest and can reduce stability.

Environment: Avoid exposing foam to fluids or moisture for prolonged periods. Avoid exposing the backrest to extreme cold or heat. Never place backrest near open flame.

Instability/Fall risk: Never place other products or materials (except those specified in this manual) between the user and the backrest or underneath the backrest, doing so, will cause instability and increase the risk of injury from a fall.

Positioning of backrest: Always refer to the orientation indicators, located on the backrest and the cover, to properly position backrest on wheelchair. Proper positioning is essential to ensure maximum comfort and pressure relief.


Skin inspection: Backrest should be tested before prolonged use. To avoid the appearance of skin redness due to pressure points, Physipro Inc. recommends shifting your weight as frequently as possible and performing a pressure relief technique at least once every two (2) hours. You should regularly check (at least once a day) for skin redness especially around bony prominences. If skin redness or discoloration develops, discontinue use of the cushion. If discoloration does not disappear within thirty (30) minutes consult a Healthcare professional immediately. Be aware that no backrest can substitute the need for proper skin hygiene, a balanced diet and frequent position changes.

Transport: Crash tests have not been conducted on these products and even if the mobility aid in which it is installed has obtained the ISO-717619, it will not be maintained.

INSTALLATION

Please read all the following instructions before beginning installation. The steps required to properly install the Conventional backrest must be followed in the order that they are listed.

1 - Installing the mounting brackets on the back plate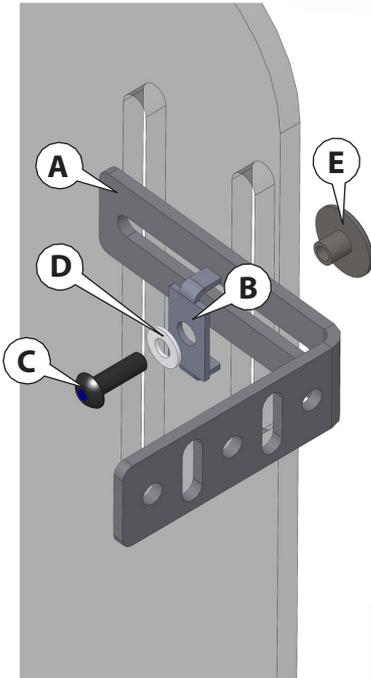

1. Firmly press mounting bracket **A** to the back plate.
2. Place bracket anchor **B** over the mounting bracket **A**.
3. Insert screw **C** through washer **D**, bracket anchor **B** and back plate and then tighten nut **E** firmly.


Note: It is important to align and level the brackets on left and right side, both horizontally and vertically.

Product should be as shown:

2 - Installing the mounting hooks on the brackets


1. Align backrest mounting hook **A** to mounting bracket **B** and determine the desired mounting depth.
2. Insert M5 screws **C** through the appropriate adjustment hole of the mounting hook and the bracket and then place washers **D** and nuts **E** on screws **C**.
3. Tighten nuts **E** firmly.
4. Repeat process 1 to 3 for all four mounting hooks.

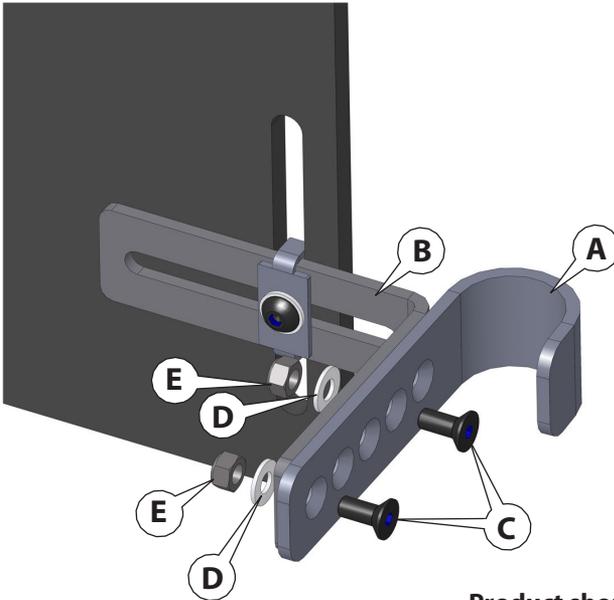
Product should be as shown:


3 - Installing the fixed stabilizers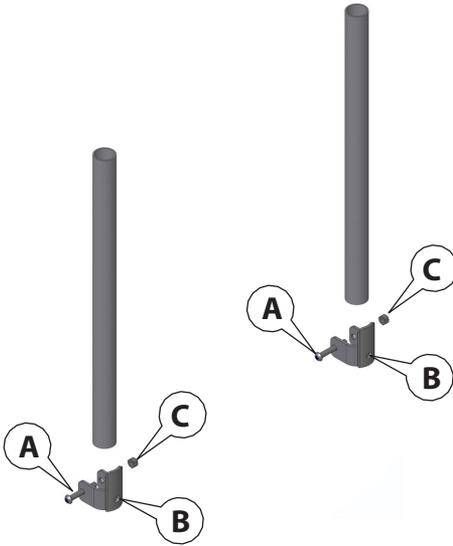

1. Loosen M6 clamping screws **A** of each stabilizer **B**.
2. Place stabilizers **B** on the backposts at the desired height.
3. Reinsert M6 screws **A** and firmly tighten nuts **C** on each stabilizer **B**.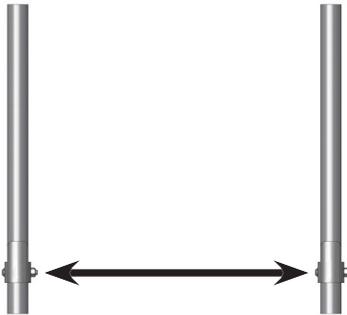


Aligned stabilizer

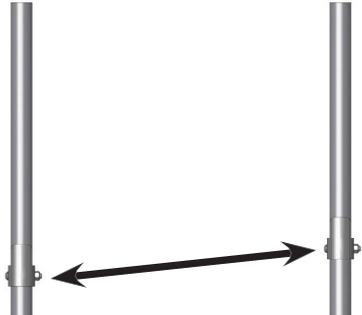
Misaligned stabilizers

4 - Installing the swing-away stabilizers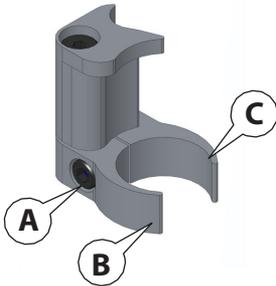

1. Loosen M5 screw **A** to extend clamps **B** and **C**.

2. Place stabilizers on the backposts at desired height.

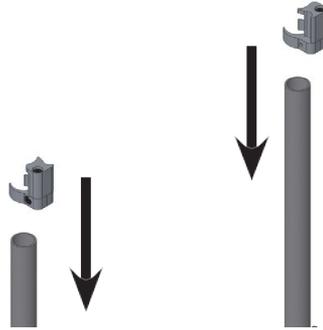3. Tighten M5 screw **A** to secure clamps **B** and **C**.

Note – Like the fixed stabilizers, it is important to align and level the swing-away stabilizers, both horizontally and vertically.


4. Loosen screws **D** and pivot the stabilizers outwards before installing the back plate.

Product should be as shown:


5 - Installing the back plate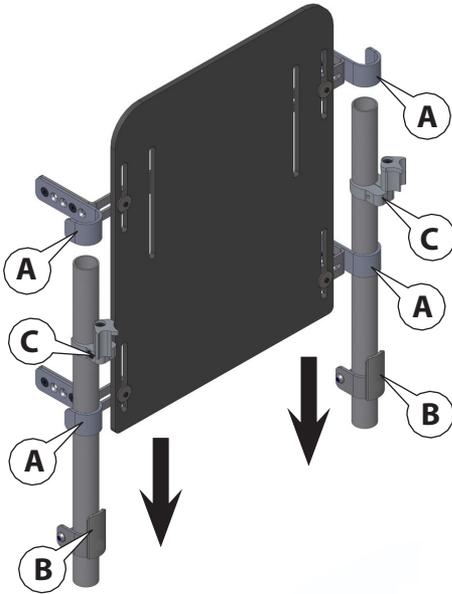

1. Place the mounting hooks **A** on the backposts.
2. Slide back plate downward till mounting hooks **A** rest on stabilizers **B** and **C**.

3. Pivot the swing-away stabilizers **C** inwards and tighten screws **D** to secure back plate in place.

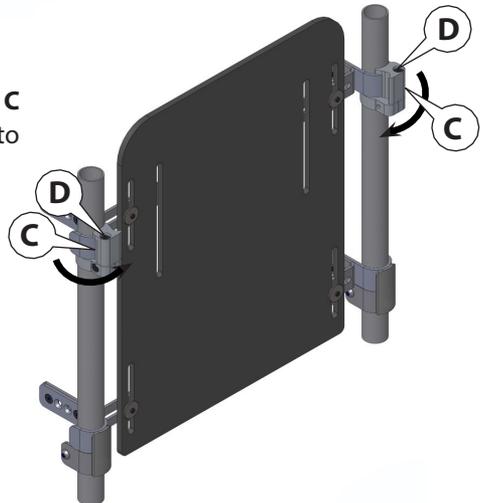MAINTENANCE

Regular cleaning eliminates the accumulation of dirt and grime and may help extend the life of your backrest. It is recommended to visually inspect the backrest during the cleaning process.

Maintenance Guide

Upon receiving your backrest, it is important to complete all the inspections listed in the Maintenance Guide to ensure the proper function and condition of the backrest and mounting hardware.

Checklist		Weekly	Montly
Backrest	• Inspect components for rust or damage.		X
	• Make sure the backrest is not deformed.		X
Hardware	• Verify the solidity and condition of mounting hardware.	X	
	• Ensure swing-away stabilizers operates properly.	X	
Laterals	• Make sure that the laterals are fastened properly.	X	
	• Inspect components for rust or damage.		X

Cleaning the outer cover:

1. Remove cover from backrest and fasten the zipper.
2. Machine wash in cold water on the gentle cycle or hand wash with a mild detergent.
3. Allow to air dry completely.
4. Reinstall cover on the foam base.

WARNING

Do not iron.

Cleaning the back plate: The aluminum or plastic back plate can be cleaned with warm water and a mild liquid detergent. No other maintenance is required.

Cleaning the backrest padding:

1. Remove outer cover and incontinence cover, if necessary.
2. Wipe foam padding lightly with a damp cloth, without soap or detergent.
3. Use a clean dry cloth to gently press on padding to absorb excess moisture, then let air dry completely.
4. Reinstall cover(s) on the backrest.

WARNINGS

*Do not use soap.
Do not submerge in water.
Do not use washer or dryer to clean
the backrest padding.*

Storage

Backrest must be stored in a dry and secure area.

WARNINGS

*Never store backrest in direct sunlight or expose to excessive humidity or heat.
Never place backrest near sharp objects or in an area where it may be struck.*

Life span

The estimated life span of your backrest is three (3) years.

Disposal of Medical Devices

Do not burn or throw away medical devices outside of designated areas.

ENGLISH

WARRANTY

The warranty periods for the parts and components of the Conventional Backrest are as stated:

- Backrest: Twenty-four (24) months
- Lateral supports : Twelve (12) months
- Outer cover: Six (6) months

This limited warranty covers defects in material and workmanship on all the standard or optional parts and components of this product, from the date of shipment, for the duration stated above.

Parts and components that are replaced or repaired under one of these warranties will remain subject to the warranty for the remainder of the warranty period. Physipro Inc. agrees to repair or replace, free of charge, any defective part or component for the full duration of the warranty period.

- This warranty does not apply to:**
- a) Products that have been neglected or have not been maintained and stored in accordance with the recommendations of the manufacturer.
 - b) Products that were intentionally or accidentally altered, or products that have been damaged through repairs or modifications made without the prior written consent of Physipro Inc. or of an authorized retailer.
 - c) Normal wear and tear or damages caused by the user such as punctures, tears, or burns.

Warranty service

If a defect is detected during the warranty period, please contact the retailer from whom you purchased your Physipro Inc. product as soon possible. Please refrain from using your product until an inspection has been made by a qualified technician.

In the event you do not receive satisfactory warranty service, please write directly to Physipro Inc. at the address listed below or send us an e-mail at order@physipro.com, please include the retailer's name and address, a brief description of the defect, the invoice date, and the products serial number, if applicable. DO NOT RETURN product prior to receiving a written return authorization from Physipro Inc.

The defective unit or parts must be returned for warranty inspection within thirty (30) days of the return authorization date at the following address:

Physipro Inc.
370, 10th Avenue South, Sherbrooke, Quebec, J1G 2R7

Exclusions and limitations

Compliance with the safety precautions and recommendations mentioned in this manual is crucial to avoid injury to the user or other persons. Inadequate or inappropriate adjustments will increase the risk of accident. It is important to thoroughly read the manual before making any adjustments. Physipro Inc. will not be held responsible for bodily injury or property damage caused by misuse or incorrect adjustments made to its product by an unqualified person.

Physipro Inc. disclaims all liability for damages that occur during transportation or are the result of situations beyond its control.

The foregoing warranty is exclusive and in lieu of all other expressed warranties.

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