PHYSIPRO[®]

CONTOURED ARM SUPPORT

Owner's Manual

This booklet contains important information about this product.
Please give to the owner upon delivery.

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INTRODUCTION

Physipro Inc. is proud to count you among its customers and would like to thank you for the confidence you have shown by purchasing our product.

This owner's manual was created to provide you with all the information needed to allow you to use our product in a safe and optimal way. For all adjustments and settings, we strongly recommend you contact your distributor.

Physipro Inc., relieves itself of all liability should physical injury or property damage occur due to the lack of maintenance or misuse of our product, or from modifications made to the product without the prior written consent of Physipro Inc.

PRODUCT INFORMATION

Date of purchase: _____

Supplier: _____

Address: _____

Telephone: _____

The product presented in this manual may not be an exact representation of your product. However, the instructions provided in this manual are still valid.

Physipro Inc. reserves the right to modify or change this manual without notice.

TABLE OF CONTENT

DESCRIPTION.....	3
COMPOSITION.....	3
TECHNICAL SPECIFICATIONS.....	3
RECOMMENDATIONS.....	4
WARNINGS.....	4
INSTALLATION.....	5
Tray slide rail.....	5
Tilting mounting clamp.....	5
Swing-away hand rest mechanism (option).....	6
Forearm strap (option).....	6
MAINTENANCE.....	7
WARRANTY.....	8
FRANÇAIS.....	9

DESCRIPTION

The Contoured arm support comfortably surrounds and supports the forearm and protects the affected upper limb. This arm support can be equipped with a swing-away hand rest mechanism and a forearm strap. As an option, a gel layer can be added for additional comfort and skin protection.

Available in standard and non-standard dimensions. The Contoured arm support can be installed on a tray slide rail or with a tilting mounting clamp.

This item meets the requirements of California 116 & 117, NFPA 701, CE marking and NF EN1021-1, EN1021-2 norms.

COMPOSITION

Rigid structure

¼-inch ABS

Padding

¼-inch Neoprene

2-inch Velcro

Lycra

¼-inch Pebble Gel (option)

Mounting hardware options

Tray slide rail (available in ABS or Aluminum)

Tilting mounting clamp

Optional components

Swing-away hand rest mechanism

Forearm strap

TECHNICAL SPECIFICATIONS

Size	Forearm Length		Forearm Width		Forearm Height		Hand Width		Hand Length	
	cm	in	cm	in	cm	in	cm	in	cm	in
Small	20	8	10	4	5	2	15	6	16,5	6½
Medium	29	11½	10	4	5	2	15	6	16,5	6½
Large	35,5	14	10	4	5	2	15	6	16,5	6½
Non-Standard	Made-to-measure according to the customer's specific requirements									

RECOMMENDATIONS

To ensure the longevity of your upper limb support, the following precautions should be applied.

- Before use, it is important to verify the quality and condition of the upper limb support's components.
- Physipro Inc. recommends that a health care professional such as a doctor or therapist, experienced in seating and positioning, be consulted to determine, and install the required upper limb support and make any necessary adjustments.
- Clean regularly and inspect the upper limb support's components periodically.
- Remove any foreign object (crumbs, debris, etc.) from the upper limb support's surface.

It is recommended to consult your doctor or therapist in the month following the installation of a new upper limb support to make sure it is efficient.

WARNINGS

- Physipro Inc. upper limb supports are reserved exclusively for use on mobility aids (manual or power wheelchairs).
- Always use the provided mounting hardware to install your upper limb support.
- Never push or lift wheelchair with an upper limb support, doing so may cause the device to detach from the wheelchair which may result in damage or injury. Always lift wheelchair with the rigid parts of the frame.
- Installing a new product on a wheelchair may affect the wheelchair's center of gravity and may cause the wheelchair to become unstable. To ensure optimal stability of your wheelchair after the addition of a new component, always assess for the potential need for anti-tips or amputee axle adapter brackets to be added or adjusted.
- Before use, let the upper limb support get to room temperature to limit discomfort caused by excessively hot or cold surfaces.
- Avoid exposing your upper limb support to extreme cold or heat and avoid prolonged contact to fluids or moisture. Never place near open flame.
- Crash tests have not been conducted on this product and even if the mobility aid in which it is installed has obtained the ISO-717619, it will not be maintained.

INSTALLATION


The Contoured Arm Support is preassembled on the mounting hardware of your choice. The following instructions explain how to adjust the position of the arm support according to your selected mounting hardware.

Tray slide rail

Required tool: 4mm Allen key

Adjust the depth of the arm support

1. Loosen screw **A**.
2. Move the tray slide rail forward or backward until the arm support is properly positioned.
3. Firmly tighten screw **A**.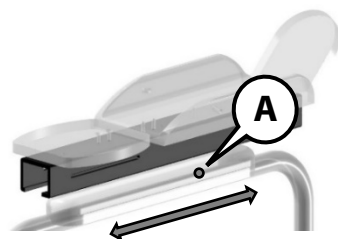


Tilting mounting clamp

Required tool: 4mm Allen key


Adjust the depth and inclination of the arm support

1. Slightly loosen both screws **B**.
2. Move the tilting mounting clamp forward or backward on the armrest tubing until the desired depth is obtained.
3. Adjust the inclination of the arm support.
4. Firmly tighten screws **B**.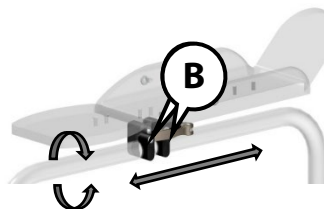

Adjust the horizontal position of the arm support

1. Release lever **C**.
2. Pivot the arm support towards the left or right.
3. When the desired position is obtained, engage the lever **C**.

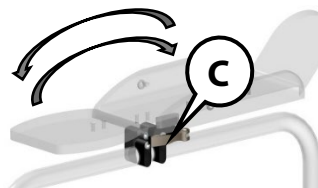Removing the contoured arm support

1. Release lever **C**.
2. Lift the arm support to remove.

Swing-away hand rest mechanism (option)

The simple to use mechanism allows the hand rest to be lowered to facilitate access to a table and allows the occupant to perform hand and wrist exercises to improve mobility. The swing-away hand rest mechanism is pre-assembled on the mounting hardware of your choice

How to use the swing-away hand rest mechanism

1. Gently pull the swing away hand rest forward to release swing-away mechanism.
2. Let the hand rest pivot downwards.
3. To reinstall, lift the hand rest upwards and gently push back until the mechanism engages.

Forearm strap (option)

This padded forearm strap prevents the upper limb from slipping out of the Contoured arm support. Made of a soft detachable neoprene pad, that can be accurately positioned over the forearm and adjustable nylon straps with a Velcro fastener, the forearm strap comfortably holds the arm in place. The forearm strap is pre-assembled directly on the Contoured arm support.

MAINTENANCE

Regular cleaning eliminates the accumulation of dirt and grime and helps extend the life of your upper limb support. It is recommended to visually inspect all the components of the upper limb support during the cleaning process.

A weekly cleaning is recommended.

Cleaning the contoured arm support

1. Remove the padding from the rigid structure.
2. With a dry cloth, dust off the rigid structure surface before cleaning.
3. Gently wipe the structure with a cloth dampened with a mild soap or neutral detergent and dry-off with a clean cloth.
4. Remove all debris on the padding surface and between seams and folds.
5. With a mix of water and a soft soap or neutral detergent, gently wipe clean the padding.
6. Remove excess moisture by lightly patting the surfaces with a clean cloth and then let air dry completely.
7. Fasten the padding to the rigid structure with the Velcro fastener.

WARNINGS

- Do not submerge in water
- Never machine wash or dry.

Storage

Upper limb supports must be stored in a dry and secure area.

WARNINGS

- Never store your upper limb support in direct sunlight, near a heat source or in a high-humidity environment.
- Never place your upper limb support near sharp objects or in an area where it may be struck by an object.

Life span

The estimated life span of your upper limb support is three (3) years.

Disposal of Medical Devices

Do not burn or throw away medical devices outside of designated areas.

WARRANTY

The warranty periods for the parts and components of the Contoured arm support are as stated:

Rigid structure (ABS): 36 months
Standard components: 12 months
Optional components: 12 months

This limited warranty covers defects in material and workmanship on all the standard or optional parts and components of this product, from the date of shipment, for the duration stated above.

Parts and components that are replaced or repaired under one of these warranties will remain subject to the warranty for the remainder of the warranty period. Physipro Inc. agrees to repair or replace, free of charge, any defective part or component for the full duration of the warranty period.

This warranty does not apply to:

- a) Products that were misused, neglected or have not been maintained and stored in accordance with the recommendations of the manufacturer.
- b) Products that were intentionally or accidentally altered, or products that have been damaged through repairs or modifications made without the prior written consent of Physipro Inc. or of an authorized retailer.
- c) Normal wear and tear or damages caused by the user such as punctures, tears, or burns.

Warranty service

If a defect is detected during the warranty period, please contact the retailer from whom you purchased your Physipro Inc. product as soon possible. Please refrain from using your product until an inspection has been made by a qualified technician.

In the event you do not receive satisfactory warranty service, please write directly to Physipro Inc. at the address listed below or send us an e-mail at **order@physipro.com**, please include the retailer's name and address, a brief description of the defect, the invoice date, and the products serial number, if applicable. **DO NOT RETURN** product prior to receiving a written return authorization from Physipro Inc. The defective unit or parts must be returned for warranty inspection within thirty (30) days of the return authorization date at the following address:

Physipro Inc.
 370, 10th Avenue South, Sherbrooke, Quebec, J1G 2R7

Exclusions and limitations

Compliance with the safety precautions and recommendations mentioned in this manual is crucial to avoid injury to the user or other persons. Inadequate or inappropriate adjustments will increase the risk of accident. It is important to thoroughly read the manual before making any adjustments. Physipro Inc. will not be held responsible for bodily injury or property damage caused by misuse or incorrect adjustments made to its product by an unqualified person.

Physipro Inc. disclaims all liability for damages that occur during transportation or are the result of situations beyond its control.

The foregoing warranty is exclusive and in lieu of all other expressed warranties.

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