PHYSIPRO®

3D-MOLDED SEATING

Owner's Manual

This booklet contains important information about this product.
Please give to the owner upon delivery.

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INTRODUCTION

Physipro Inc. is proud to count you among its customers and would like to thank you for the confidence you have shown by purchasing our product.

This owner's manual was created to provide you with all the information needed to allow you to use our product in a safe and optimal way. For all adjustments and settings, we strongly recommend you contact your distributor.

Physipro Inc., relieves itself of all liability should physical injury or property damage occur due to the lack of maintenance or misuse of our product, or from modifications made to the product without the prior written consent of Physipro Inc.

For Physipro Inc., your satisfaction is our highest priority.

PRODUCT INFORMATION

Date of purchase: _____

Supplier: _____

Address: _____

Telephone: _____

The product presented in this manual may not be an exact representation of your product. However, the instructions provided in this manual are still valid.

Physipro Inc. reserves the right to modify or change this manual without notice.

TABLE OF CONTENT

DESCRIPTION.....	3
COMPOSITION.....	3
WARNINGS	4
RECOMMENDATIONS.....	5
SEAT CUSHION INSTALLATION	5
BACKREST MOUNTING SYSTEM INSTALLATION	6
BACKREST CUSHION INSTALLATION	9
MAINTENANCE.....	10
WARRANTY.....	11
FRANÇAIS	12

3D-MOLDED SEATING

The 3D-Molded Seating system was developed to pinpoint the postural needs of each client.

With the use of an advanced digitalizing system, a 3D image of the client's spinal and orthopedic anatomy is captured to create a custom-made seat and/or backrest cushion.

Each molded cushion accurately encompasses the contours of the client's body while in a seated position, optimizing postural stability and comfort.

This item meets the requirements of California 116 & 117, NFPA 701, CE marking and NF EN1021-1, EN1021-2 norms.

COMPOSITION

Foam Base Options

Injected urethane foam

- Soft density - 3.4
- Firm density - 4.5

Top Layer Options

- Extra-Soft Memory Foam
- Soft Memory Foam
- Viscoelastic Foam

Seat Cover

- Stretch cover with zipper

Available Backrest Covers

- Stretch cover with zipper
- Stretch slipcover

Available Inner Covers (for Seat and Backrest)

- Waterproof transparent cover with velcro

Backrest Options

- Glued-on ABS plate
- Stimulite® Insert
- Roho® Insert
- Lateral reinforcements
- Clearance for harness installation
- Ventilation holes
- Mounting systems (installed on backposts)

Seat Options

- Stimulite® Insert
- Roho® Insert
- Gel Inserts:
 - Pebble gel
 - Cubed gel
 - Silicone gel
 - Geo-Matrix™ gel (Available in Quebec only)
- Lateral reinforcements

Waterproofing

A waterproofing spray coating can also be applied to protect the cushion's surface from damage sustained by exposure to liquids.

WARNINGS

Consult a healthcare professional: Physipro™ recommends that a health care professional such as a doctor or therapist, experienced in seating and positioning, be consulted to determine, and install the required seating system and make any necessary adjustments.

Center of gravity: Installing a cushion on a wheelchair may affect the wheelchair's center of gravity and may cause the wheelchair to tip backwards, potentially resulting in injury. To ensure optimal stability of your wheelchair after the addition of a new cushion, always assess for the potential need for anti-tips or amputee axle adapter brackets to be added or adjusted to increase stability.

Cushion dimensions: The dimensions of the cushion must be adapted to the user and the wheelchair, if not, the cushion's benefits will be reduced or eliminated.

Cushion positioning: Always verify that the cushion is properly positioned on the wheelchair. Proper positioning is essential to ensure maximum comfort and pressure relief.

Cushion and cover compatibility: The cushion cover must be well-fitted to prevent the formation of skin lesions. An oversized or undersized cover will reduce or eliminate the effectiveness of the cushion and can create an unstable surface, which can lead to a fall.

Instability/Fall risk: Never place other products or materials (except those specified in this manual) between the user and the cushion or underneath the cushion, doing so, will cause instability and increase the risk of injury from a fall.

Environment: Avoid exposing foam to fluids or moisture for prolonged periods. Avoid exposing the cushion to extreme cold or heat. Never place cushion near open flame.

Skin inspection: Cushion should be tested before prolonged use. To avoid the appearance of skin redness due to pressure points, Physipro™ recommends shifting your weight as frequently as possible and performing a pressure relief technique at least once every two (2) hours. You should regularly check (at least once a day) for skin redness especially around bony prominences. If skin redness or discoloration develops, discontinue use of the cushion. If discoloration does not disappear within thirty (30) minutes consult a healthcare professional immediately. Be aware that no cushion can substitute the need for proper skin hygiene, a balanced diet and frequent position changes.

Transport: Crash tests have not been conducted on these products and even if the mobility aid in which it is installed has obtained the ISO-717619, it will not be maintained.

RECOMMENDATIONS

Before use, verify the quality and condition of your molded seat and/or backrest cushion's components and the mounting hardware provided.

To ensure your cushion provides optimal protection and comfort and to extend your product lifespan, the following precautions should be applied.

- Physipro™ molded seat and backrest cushions are reserved exclusively for use on mobility aids (manual or power wheelchairs), do not use on other surfaces.
- Before installation, make sure the cushion is compatible with the wheelchair dimensions.
- Cushion must be installed on a good quality flat wheelchair surface.
- Use only the installation hardware provided and follow the installation process described in this manual.
- Never place a towel, a blanket, a folded bed sheet or any other material between the user and the cushion, this will diminish the cushion efficiency and increase the risk of developing skin lesions.
- Verify height of armrest. Adjustments should be made, if needed.
- Before use, let the cushion get to room temperature to limit discomfort caused by excessively hot or cold surfaces.
- Never place molded seat and/or backrest cushion near sharp object.
- Remove any foreign object (crumbs, debris, etc.) from cushion surface.
- Clean on a regular basis and visually inspect all components of the cushion periodically.

It is recommended to consult your therapist in the month following the installation of a new cushion to make sure it is efficient.

SEAT CUSHION INSTALLATION

Your molded seat cushion is easily installed with the use of velcro strips.

Two self-adhesive velcro strips are attached to two fixed velcro strips, located on the bottom of the cushion's outer cover.

Before installation:

- Make sure the seat pan is clean and dry.
 - Inspect the outer cover to ensure that the stitches are aligned with the cushion's edges.
1. Remove the protective film of the self-adhesive velcro strips.
 2. Position the cushion on the seat pan.
 3. Press down firmly on the cushion to ensure that both self-adhesive velcro strips have properly adhered to the seat pan.

BACKREST MOUNTING SYSTEM INSTALLATION

According to the selected mounting system your backrest can be installed in front of the backpost (Standard mounting system) or behind the backpost (Built-in mounting system).

Both systems include the following hardware:

4 Slot brackets

A variety of lengths allow for different mounting depths.

4 Mounting hooks

A variety of lengths allow for different mounting depths.

Mounting plate

4 Bracket anchors


4 Fixed stabilizers


Step 1: Installing the slot brackets

Required tool: 4mm Allen key

1. Place slot bracket **A** over the backrest mounting plate.
2. Put bracket anchor **B** over slot bracket **A**.
3. Position end cap **C** on the opposite side of the mounting plate.
4. Place washer **D** on screw **E** and insert screw through bracket and into the threaded end cap **C**.
5. Adjust the bracket vertically and horizontally and then firmly tighten screw **E**.
6. Repeat process for all four slot brackets.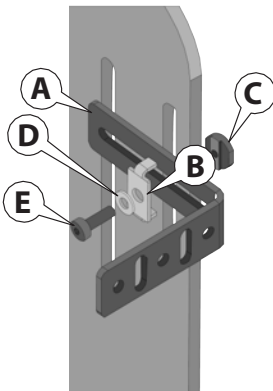

Product should be as shown:

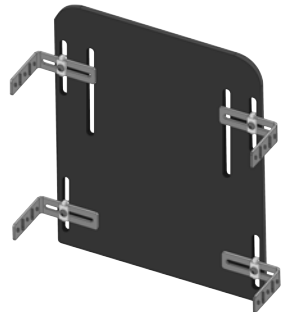
Note: It is important to align the right and left sides at the same level and to ensure that the vertical alignment is the same on each side.


Step 2: Installing the mouting hooks

Required tool: 3 mm Allen key

1. Align mounting hook **A** with the slot bracket **B** at the desired depth.
2. Insert screws **C** through the positionig holes of the mouting hook and the bracket slot.
3. Install washers **D** and then tighten nuts **E** firmly.
4. Repeat process for all four mounting hooks.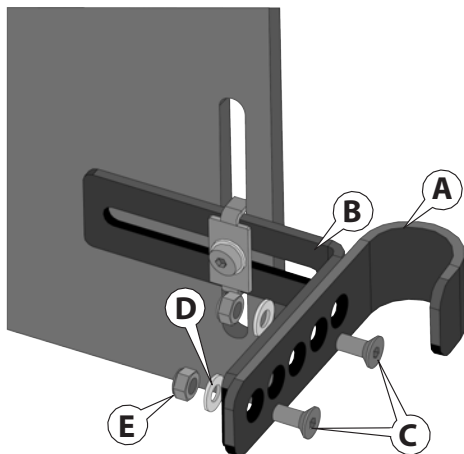


Product should be as shown:

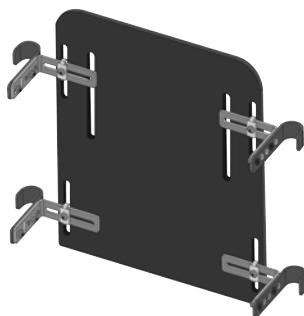Step 3: Installing the lower fixed stabilizers

Required tool: 4mm Allen Key

1. Unfasten screw **A** and nut **C**.
2. Position stabilizer **B** on the backpost as illustrated.
3. Reinsert screw **A** and place nut **C** on the end of screw **A**.
4. Position stabilizer at the desired height and firmly tighten screw **A** and nut **C**.
5. Repeat the process for the second stabilizer.

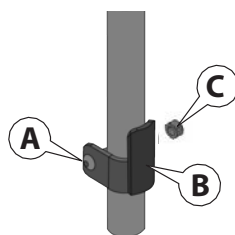
Note: Make sure both lower stabilizers are vertically aligned.


Step 4: Installing the mounting plate on the backposts

1. Place the mounting hooks **A** on the backposts.
2. Slide mounting plate assembly downwards until the lower mounting hooks **A** rest on the lower fixed stabilizers **B**.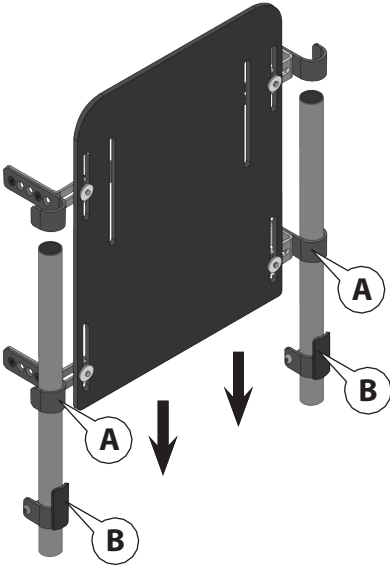


Product should be as shown:

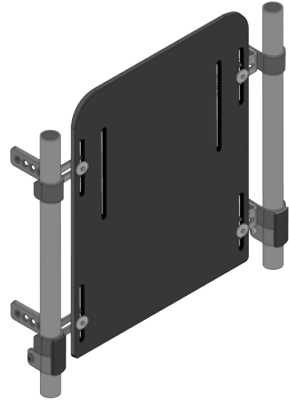**Step 5: Installing the upper fixed stabilizers**

Required tool: 4mm Allen Key

1. Unfasten screws **A** and nuts **C** of both upper stabilizers.
2. Position upper stabilizers **B** on the backposts as illustrated.
3. Slide the upper fixed stabilizers **B** downwards until they rest on the upper mounting hooks **D**.
4. Firmly tighten screws **A** and nuts **C**.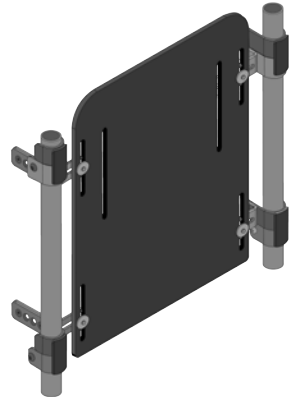

BACKREST CUSHION INSTALLATION

According to the selected cushion installation, your backrest can be installed with velcro or with the ABS plate and removable screws system.

Installing backrest cushion with velcro

Your molded backrest cushion is easily installed with the use of velcro strips.

Self-adhesive velcro strips are attached to fixed velcro strips, located on the back of the cushion's cover.

Before installation:

- Make sure the backrest mounting plate is clean and dry.
- Inspect the cover to ensure that the stitches are aligned with the cushion's edges.


1. Remove the protective film of the self-adhesive velcro strips.
2. Position the backrest cushion on the mounting plate.
3. Press the backrest cushion firmly on the mounting plate to ensure that the self-adhesive velcro strips have properly adhered to the mounting plate.

Installing backrest with the glued-on ABS plate system

1. Position the molded backrest cushion with ABS plate **A** on the backrest mounting plate **B**.

Note: Make sure the mounting holes are aligned.

2. Put washers **C** on threaded knobs **D**.
3. Insert threaded knobs **D** into the mounting holes.
4. Firmly turn all 4 threaded knobs **D** to securely fasten the backrest cushion.

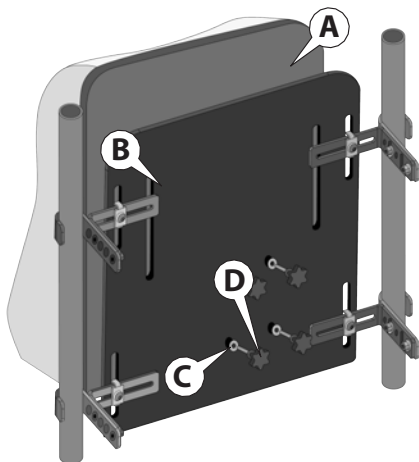MAINTENANCE

Regular cleaning eliminates the accumulation of dirt and grime and may help extend the life of your cushion. It is recommended to visually inspect the components of the cushion during the cleaning process.

Cleaning the outer cover

1. Remove cover from the foam base and fasten the zipper.
2. Machine wash in cold water on the gentle cycle or hand wash with a mild detergent.
3. Allow to air dry completely.
4. Reinstall cover on the foam base.

WARNINGS

No ironing required.

Cleaning the incontinence cover

Hand wash with soapy water and use a clean dry cloth to gently wipe off excess moisture, then let air dry completely before reinstalling.

Cleaning the cushion

1. Remove the outer cover and the incontinence cover.
2. Wipe lightly with a damp cloth, without soap or detergent.
3. Use a clean dry cloth to gently press on cushion to absorb excess moisture, then let air dry completely.
4. Reinstall cover(s).

WARNINGS

Do not use soap.

Do not submerge in water.

Never place foam base in washer or dryer.

Storage

Physipro™ cushions should be stored on a flat surface, in a dry area and out of direct sunlight.

Disposal of Medical Devices

Do not burn or throw away medical devices outside of designated areas.

WARRANTY

The warranty periods for the following part(s) and component(s) of the 3D-Molded seat and/or backrest are as stated below:

- Foam base, gels and hardware: Two (2) years
- Covers and spray coating: Six (6) months

These limited warranties cover defects in material and workmanship on all standard or optional parts and components of this product, for the periods indicated above, starting from the date of shipment.

Parts and components that are replaced or repaired under one of these warranties will remain subject to the warranty for the remainder of the warranty period. Physipro Inc. agrees to repair or replace, free of charge, any defective part or component for the full duration of the warranty period.

This warranty does not apply to:

- a) Products that were misused, neglected or have not been maintained and stored in accordance with the recommendations of the manufacturer.
- b) Products that were intentionally or accidentally altered, or products that have been damaged through repairs or modifications made without the prior written consent of Physipro Inc. or of an authorized retailer.
- c) Normal wear and tear or damages caused by the user such as punctures, tears, or burns.

Warranty service

If a defect is detected during the warranty period, please contact the retailer from whom you purchased your Physipro Inc. product as soon possible. Please refrain from using your product until an inspection has been made by a qualified technician.

In the event you do not receive satisfactory warranty service, please write directly to Physipro Inc. at the address listed below or send us an e-mail at order@physipro.com, please include the retailer's name and address, a brief description of the defect, the invoice date, and the products serial number, if applicable. DO NOT RETURN product prior to receiving a written return authorization from Physipro Inc.

The defective unit or parts must be returned for warranty inspection within thirty (30) days of the return authorization date at the following address:

Physipro Inc.

370, 10th Avenue South, Sherbrooke, Quebec, J1G 2R7

Exclusions and limitations

Compliance with the safety precautions and recommendations mentioned in this manual is crucial to avoid injury to the user or other persons. Inadequate or inappropriate adjustments will increase the risk of accident. It is important to thoroughly read the manual before making any adjustments. Physipro Inc. will not be held responsible for bodily injury or property damage caused by misuse or incorrect adjustments made to its product by an unqualified person. Physipro Inc. disclaims all liability for damages that occur during transportation or are the result of situations beyond its control.

The foregoing warranties are exclusive and in lieu of all other expressed warranties.

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